





















alkaline battery x 12 pcs

inner diameter: 3.2 cm

# **ABOUT COCK RINGS**

Please read this information carefully before you use a cock ring for the satisfied with the quality of their erection or men whose erection is not persistent tight enough for sexual activity.

### Diagnosis of other conditions

injuries of the nerve fibers, alcoholism, drugs, smoking and taking medications can be the cause of erectile dysfunction.

## Instructions for use

1. The penis ring is slipped over the penis before arousal. For the best

- permanent damage to the penis.
- 2. Penis rings can be worn not only on the glans or on the penis but als
- back to the body. As result of the erection is faster, stronger and longer

- The ring can improve and reinforce the naturally erection.
- erection decreases. Then you can first pull out the penis and then the

Warnings - ATTENTION!
Please note that the ring should be worn no longer than 30 minutes. Prolonged use of the ring (i.e. without removal) may cause permanent injury to the penis.

Do not fall asleep while wearing the penis ring

Please wait at least 60 minutes between uses, as more frequent use may increase the risk of injury to the penis

Do not use the ring under the influence of alcohol or drugs because such use may impair the user's judgment

application please stop the use of the ring immediate

area of the penis because you do not feel the pain in the case of Do not use the ring if you have decreased hand strength because

this may make removal of the ring difficult. Please consult a physician if any complications occur and do not

use the ring if you continue to experience complications.

Do not give the ring at third parties to avoid diseases and

Keep out the ring of reach of children!

under warranty terms and conditions.

Misuse of the penis ring may cause bruising, painful injury or

The manufacturer is not responsible if you use the product

## PRODUCT WARRANTY

If a product fails to work due to manufacturing defects, we will fully replace it for the purchased item. Warranty claims must be made within 1 year of the date of purchase. The warranty only covers manufacturing defects. Normal wear and tear, negligent handling or misuse, do not fall

# CLEANING & SAFETY

Clean your VĕDO™ toy before and after each use with a soft cloth, antibacterial soap and warm water. Rinse well to remove any residue. Always let toys dry completely. When not in use, remove batteries and store in a clean dry place. Do not use toy if visibly damaged. If the toy malfunctions turn it off and discontinue immediately. Do not use toy on swollen skin or areas of preexisting injuries.

VeDO™ tovs are made from 100% medical grade silicone and ABS plastic. They do not contain latex or phthalates.

PRODUCT GUIDE









